

**IMPORTANT: READ THIS USER GUIDE  
BEFORE OPENING  
BLISTER PACKAGE**




# IMPORTANT INFORMATION FOR PATIENT/CAREGIVER

## ENOXAPARIN SODIUM INJECTION

### SAFETY DEVICE DIRECTIONS FOR USE

Read this leaflet carefully before you start to use Enoxaparin Sodium Injection. Keep this information because it has important information about how to properly use this medication. These directions for use may differ from the instructions provided by your healthcare provider because instructions are different for different manufacturers. Your healthcare provider has more information and advice. Read each new leaflet that comes with a refill of your prescription because there may be new information.


Enoxaparin Sodium Injection comes as a prefilled glass syringe with attached passive safety device to prevent accidental needle sticks.

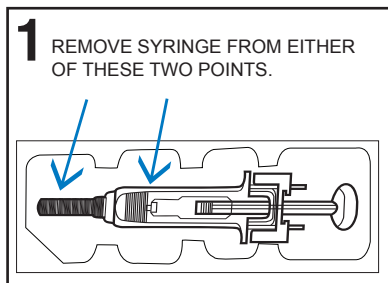

Remove the prefilled syringe from the blister packaging by peeling the lid completely off at the arrow as directed on the blister. Then gently lift the syringe straight up by grasping either the middle of the syringe or the needle cap (SEE FIGURES 1 and 2). Do not remove by pulling on the plunger as this may cause the syringe to fall apart.


View video at [www.amphastar.com/enoxaparin.html](http://www.amphastar.com/enoxaparin.html)

**DO NOT TOUCH OR MANIPULATE SAFETY DEVICE ACTIVATION PRONGS. ACCIDENTAL SAFETY DEVICE ACTIVATION WILL OCCUR MAKING THE SYRINGE UNUSABLE.**

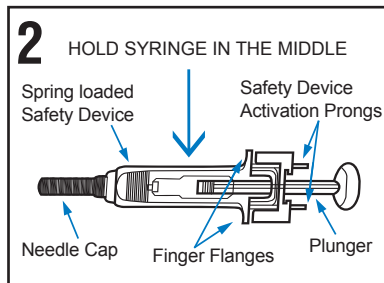


Hold the syringe in the middle, then remove needle cap by pulling straight off, to avoid bending the needle. **DO NOT** hold the syringe by the plunger when removing the needle cap, this may cause the syringe to fall apart.

If adjusting the dose is required, the adjustment must be done prior to injecting the prescribed dose. It is not necessary to expel the air bubble for 30 mg and 40 mg sizes prior to administering.

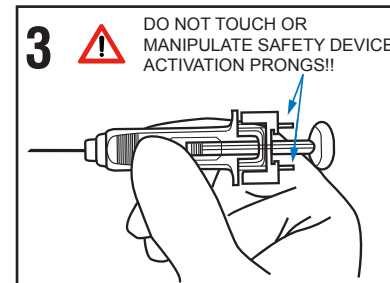


Enoxaparin Sodium Injection is administered by deep subcutaneous (SC) injection as instructed by your healthcare provider. Administration should be alternated between the left and right sides of the abdomen. Gently pinch a portion of the pre-cleansed skin to create a fold of skin and fat about 1 inch thick. The whole length of the needle should be introduced into the skin fold between the thumb and forefinger. The skin fold should be held throughout the injection.

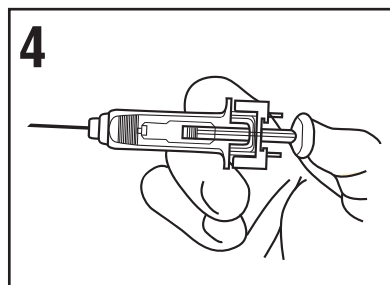

Once the full length of the needle is inserted, slowly depress the plunger while grasping the finger flanges until the entire dose is given. Keep pressure on the plunger until the needle is removed from the skin. The Passive Safety Device activates when the entire dose is given and pressure is removed from the plunger.



Once the syringe is removed from the skin, let go of the plunger to allow the safety device to cover the exposed needle.

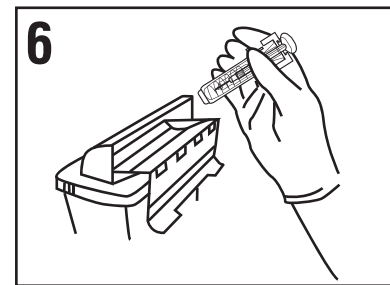

Dispose of syringe in an approved sharps container.

#### NOTE:

- The safety device should only activate once the syringe has been emptied
- Activation of the safety device must be done only after removing the needle from the patient's skin
- Do not replace the needle cap after injection
- The safety device should not be sterilized
- Activation of the safety device may cause minimal splatter of fluid. For optimal safety, activate the safety device while pointing it downwards and away from yourself and others

Full prescribing information including Boxed Warning is included in the accompanying 10-pack carton.

Report negative side effects of Prescription Drugs to FDA.  
Visit [www.fda.gov/safety/medwatch](http://www.fda.gov/safety/medwatch) or call 1-800-FDA-1088



Questions? Please call 1-800-423-4136  
Amphastar Pharmaceuticals, Inc.  
11570 Sixth Street  
Rancho Cucamonga, CA 91730  
[www.amphastar.com](http://www.amphastar.com)

06-500-02  
08/14