

**IMPORTANT: READ THIS USER GUIDE
BEFORE OPENING
BLISTER PACKAGE**

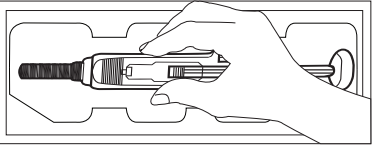
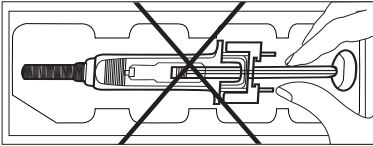
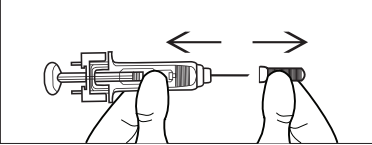
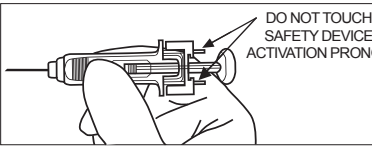
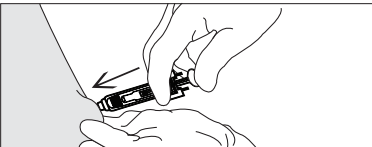
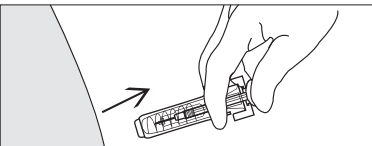
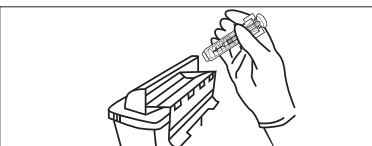


IMPORTANT INFORMATION FOR PATIENT/CAREGIVER

ENOXAPARIN SODIUM INJECTION SAFETY DEVICE DIRECTIONS FOR USE

Read this leaflet carefully before you start to use Enoxaparin Sodium Injection. Keep this information because it has important information about how to properly use this medication. These directions for use may differ from the instructions provided by your healthcare provider because instructions are different for different manufacturers. Your healthcare provider has more information and advice. Read each new leaflet that comes with a refill of your prescription because there may be new information.

Enoxaparin Sodium Injection comes as a prefilled glass syringe with attached passive safety device to prevent accidental needle sticks.

1	 <ul style="list-style-type: none">• Peel lid completely off at the arrow as indicated on the blister• Gently lift syringe straight up• View video at www.amphastar.com/enoxaparin.html <p>DO NOT TOUCH OR MANIPULATE SAFETY DEVICE ACTIVATION PRONGS. ACCIDENTAL SAFETY DEVICE ACTIVATION WILL OCCUR.</p>	 <ul style="list-style-type: none">• Do not remove syringe by pulling on plunger
2		Remove needle cover by pulling straight off of needle. DO NOT twist the needle cover or touch the needle/plunger. If adjusting the dose is required, the adjustment must be done prior to injecting the prescribed dose into the patient.
3		Administer by subcutaneous (SC) injection as instructed by your pharmacist or healthcare provider.
4		Push the plunger with slow constant pressure while grasping the finger flanges. Push the plunger all the way down as far as it will go to inject entire dose. NOTE: The passive needle guard will not activate unless the entire dose is injected.
5		Remove needle from injection site, then let go of the plunger and allow syringe to move up until the entire needle is guarded. Keep fingers in place on flanges.
6		Dispose of syringe / needle guard assembly in approved sharps container.

NOTE:

- The safety device should only activate once the syringe has been emptied
- Activation of the safety device must be done only after removing the needle from the patient's skin
- Do not replace the needle cap after injection
- The safety device should not be sterilized

Activation of the safety device may cause minimal splatter of fluid. For optimal safety, activate the safety device while pointing it downwards and away from yourself and others

Full prescribing information including Boxed Warning is included in the accompanying 10-pack carton.

Report negative side effects of Prescription Drugs to FDA. Visit www.fda.gov/safety/medwatch or call 1-800-FDA-1088

